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1. Pompa nuk thith apo thith pak

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Troubles and solutions

1. Pump suction to little or not at all

Remedy: Assemble the cover with overflow-protection again
Control the valve on the valve holder
Plug again the horn-regulator on the pump

2. The pumping handle/horn regulator are difficult to move

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Tip

Lubricating with cream, or baby oil the insulation tire and regulator after you make sterilization, makes it easy for pump to work better

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*The pump unit (A) itself must NOT be sterilized in the microwave oven. Use enough water when boiling or steam-cleaning the parts.

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All materials comply with the standards for medical products

produced by:
prodhon: **ipc**

Made in Kosova
Swiss License

www.ipc-plastics.com

baby
EASY

BPA FREE

MANUAL BREAST PUMP
with ergonomically shaped handle

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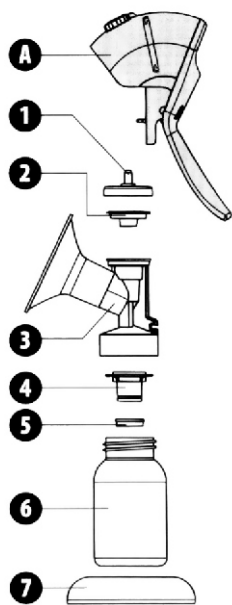
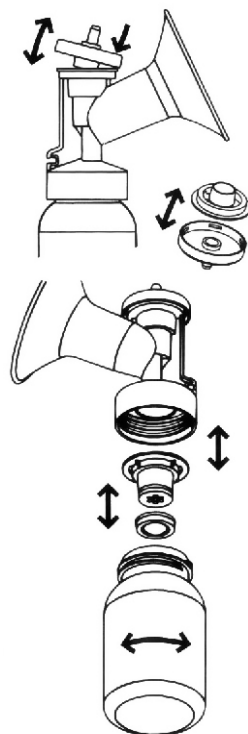
Hand-Pump parts

- A. Pump unit
1. Cover
 2. Overflow protection
 3. Suction horn
 4. Valve holder
 5. Valve
 6. Bottle
 7. Base

Manual breast pump

The ergonomic shape of the pump enables the mother to pump with one hand, relaxed and comfortable, allowing the milk to flow easily.

- Single-handed operation
- Continuous vacuum adjustment
- Adjustable (rotational) pump mechanism
- Overflow protection
- Simple to assemble and disassemble
- Horn-assembly can also be used for the electrical breast pump
- Easy to clean

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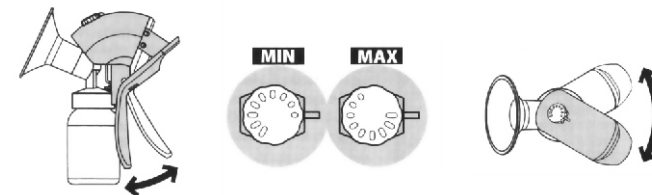
Removal of the pump unit

Turn the pump side-wise until the holder is out of the suction horn fixture (see detail). Then you can remove the pump unit.

Dismantling the horn-assembly

Push up the cover at one side with the thumb and remove the cover and the overflow-protection. Turn the bottle out of the suction horn and remove the valve and valve holder from the suction horn. Remove further the valve from the valve holder.

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Operation of the Babyeasy hand pump

The suctioning strength builds-up by operating the handle and can be regulated by the horn-regulator. Start at low vacuum (see MIN setting at the horn-regulator) and turn the horn-regulator until the vacuum level becomes comfortable. The setting MAX of the horn-regulator indicates the maximum suctioning level. The pump can be turned to either side on the horn-assembly, thus enabling a comfortable pumping.

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Use of the pump and useful tips

- Prepare yourself quietly for the pumping. Sit comfortable and relaxed. Press lightly but tight the suction horn over the nipple. Start pumping
- Pump alternating both breasts
- Should you have any questions related to the use of breast pump please contact your lactation consultant or the midwife.