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1. Pompë nuk thith apo thith pak

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Sterilize all parts before you use the horn-assambly for the first time:

Enëlarëse	Dishwasher-safe	Mikrovalë
Dishwasher-safe	Spülmaschinenfest	Microwave oven
Spülmaschinenfest	Lave-vaisselle	Microwelle
Lave-vaisselle		Micro-onde

Troubles and solutions

1. Pump suctions to little or not at all

Remedy: Assemble the cover with overflow-protection again
Control the valve on the valve holder
Plug again the horn-regulator on the pump

2. The pumping handle/horn regulator are difficult to move

Remedy: See tip below

Tip

Lubricating with cream, or baby oil the insulation tire and regulator after you make sterilization, makes it easy for pump to work better

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EN Manual breast pump

with ergonomically shaped handle

The ergonomic shape of the pump enables the mother to pump with one hand, relaxed and comfortable, allowing the milk to flow easily.

- Single-handed operation
- Continuous vacuum adjustment
- Adjustable (rotational) pump mechanism
- Overflow protection
- Simple to assemble and disassemble
- Horn-assembly can also be used for the electrical breast pump
- Easy to clean

All materials comply with the standards for medical products

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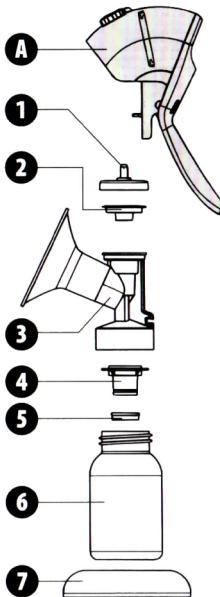
HAND-PUMP PARTS

- A. Pump unit
1. Cover
2. Overflow protection
3. Suction horn
4. Valve holder
5. Valve
6. Bottle
7. Base

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I Removal of the pump unit

Turn the pump side-wise until the holder is out of the suction horn fixture (see detail). Then you can remove the pump unit.

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
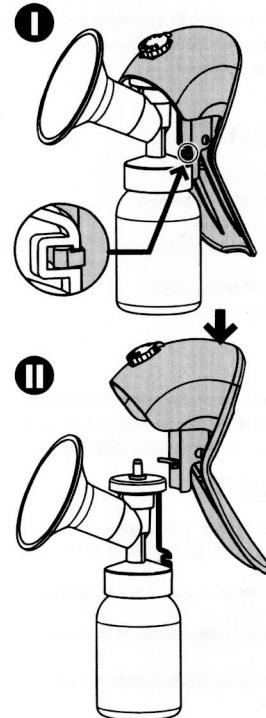
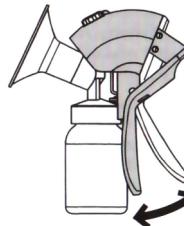
Operation of the Babyeasy hand pump

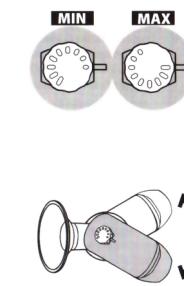The suctioning strength builds-up by operating the handle and can be regulated by the horn-regulator. Start at low vacuum (see MIN setting at the horn-regulator) and turn the horn-regulator until the vacuum level becomes comfortable. The setting MAX of the horn-regulator indicates the maximum suctioning level. The pump can be turned to either side on the horn-assembly, thus enabling a comfortable pumping.

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II Dismantling the horn-assembly

Push up the cover at one side with the thumb and remove the cover and the overflow-protection. Turn the bottle out of the suction horn and remove the valve and valve holder from the suction horn. Remove further the valve from the valve holder.

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