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## Manual breast pump

The ergonomic shape of the pump enables the mother to pump with one hand, relaxed and comfortable, allowing the milk to flow easily.

- Single-handed operation
- Continuous vacuum adjustment
- Adjustable (rotational) pump mechanism
- Overflow protection
- Simple to assemble and disassemble
- Horn-assembly can also be used for the electrical breast pump
- Easy to clean

## Tip

Lubricating with cream, or baby oil the insulation tire and regulator after you make sterilization, makes it easy for pump to work better

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**All materials comply with the standards for medical products**

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HIGH QUALITY PRODUCT

Swiss Brand  
Swiss License

**MANUAL BREAST PUMP**  
with ergonomically shaped handle

## MANUAL BREAST PUMP with ergonomically shaped handle

## MANUAL BREAST PUMP with ergonomically shaped handle

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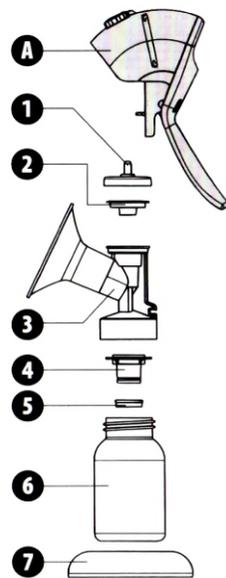
### OUR PHILOSOPHY

Children enrich our lives in the most wonderful way. In the IPC we support young parents who suddenly face new challenges and tasks. With much love and care parents from start do care that their child grow healthy.

In the IPC we would like to support parents with baby - usable and more qualitative products, that facilitate daily.

IPC is developing new permanent things. A very important part of the IPC success factor lies in the spirit of the invention and power of new discoveries of its associates. Cooperating and working closely with the engineering office in Switzerland (Trimed AG) which is specialized in the field of milk pumps, with different doctors and midwives, we apply breakthroughs and the new knowledge in the field of medicine in our products. By doing this we do give parents the assurance to act in right and just way towards their children.

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#### Hand-Pump parts

##### A. Pump unit

1. Cover
2. Overflow protection
3. Suction horn
4. Valve holder
5. Valve
6. Bottle
7. Base

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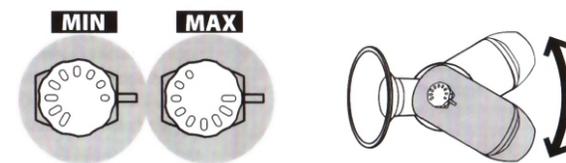
#### Operation of the Babyeasy hand pump

The suctioning strength builds-up by operating the handle and can be regulated by the horn-regulator. Start at low vacuum (see MIN setting at the horn-regulator) and turn the horn-regulator until the vacuum level becomes comfortable. The setting MAX of the horn-regulator indicates the maximum suctioning level. The pump can be turned to either side on the horn-assembly, thus enabling a comfortable pumping.

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